



EZ-ONE

ONE FOR ALL

NIC[®]

For Better Endo

EZ-ONE

ONE FOR ALL




ISO color coding



radiopaque stopper



precise depth marking

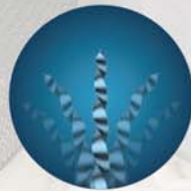


R-phase NiTi



parallelogram




rotation

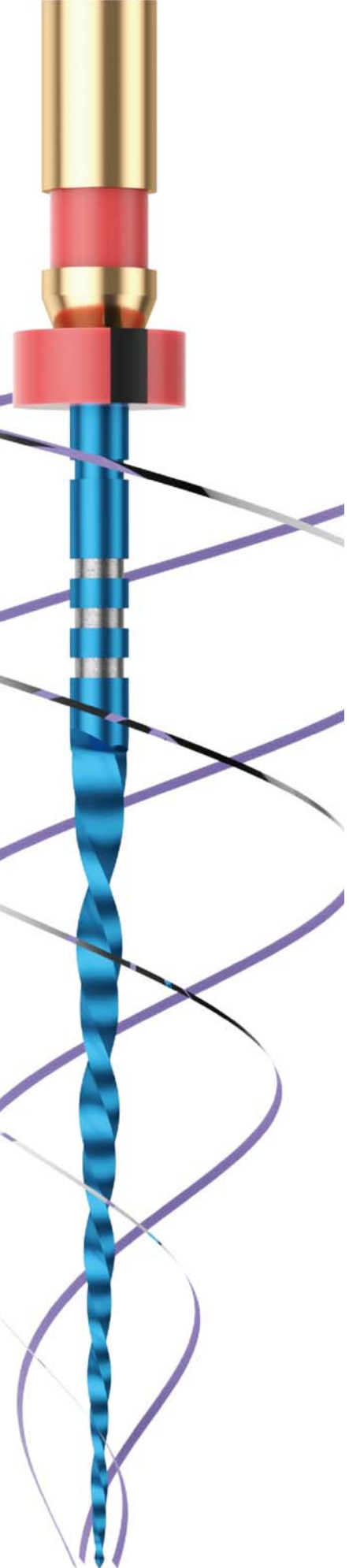

Features

- MaxTech 6.0 NiTi alloy
- Proper flexibility and strong cutting ability
- One file acts as shaper and finisher
- More space for debris removal
- Regressive taper to bring accurate cutting
- Progressive pitch reduce screw-in
- Single file technique

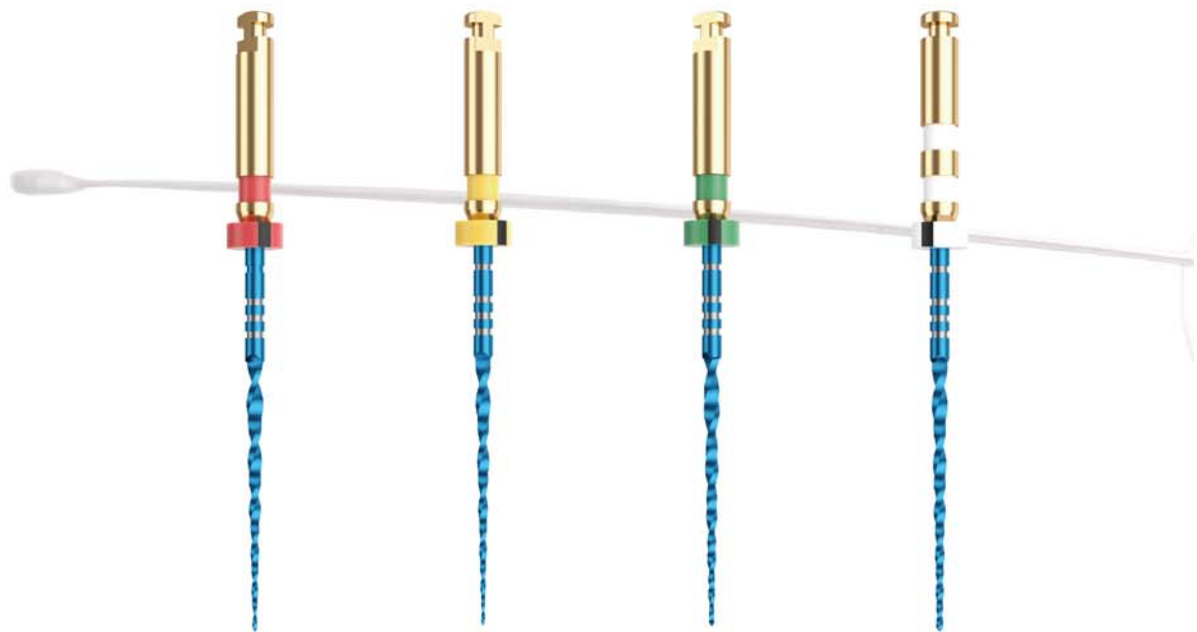

EZ-ONE	Primary	Small	Medium	Large
Size	25/.07	20/.07	35/.06	45/.05
Speed(RPM)	400	400	400	400
Torque(N.CM)	3.0	1.5	1.5	3.0

Sequence

- Scout the canal with hand use file
- Create glide path by any path function file
- Choose proper size shaping files
- Shaping the canal with the chosen EZ-ONE until it reaches working length
- After shaping and finishing is done, then choose the matched GP points for obturation.

SSK #10

T-Blue

Primary



- ◆ Small 20/.07 for narrow canals
- ◆ Primary 25/.07 for most cases
- ◆ Medium 35/.06 & Large 45/.05 for wide canals



References



		19mm	21mm	25mm	31mm
EZ-ONE single size 3pcs/blister	Small	20/.07	EZO 021 2007	EZO 025 2007	EZO 031 2007
	Primary	25/.07	EZO 021 2507	EZO 025 2507	EZO 031 2507
	Medium	35/.06	EZO 021 3506	EZO 025 3506	EZO 031 3506
	Large	45/.05	EZO 021 4505	EZO 025 4505	EZO 031 4505
EZ-ONE assorted sizes 3pcs/blister	Basic Kit (SX- Blue, T-Blue, Primary)	19/.04 17/.02 25/.07	EZO 021 A001	EZO 025 A001	EZO 031 A001
	Advanced Kit (Small, Primary, Medium)	20/.07 25/.07 35/.06	EZO 021 A002	EZO 025 A002	EZO 031 A002
EZ-CAN single size 3pcs/blister	SX- Blue	19/.04	BFU 019 1904		
	T- Blue	17/.02		BFT 021 1702	BFT 025 1702 BFT 031 1702

Clinic Cases



Endo Motor & Obturation



Small: GPW 029 020
 Primary: GPW 029 025
 Medium: GPW 029 035
 Large: GPW 029 045



Small: PPW 029 020
 Primary: PPW 029 025
 Medium: PPW 029 035
 Large: PPW 029 045



Premium manufacturer
since 2002

SUPERLINE



SHENZHEN SUPERLINE TECHNOLOGY CO., LTD.

ISO 13485:2016 CE 197 FDA

www.supline.com