


SOVA[®]


night guard

Nighttime grind protection.[™]

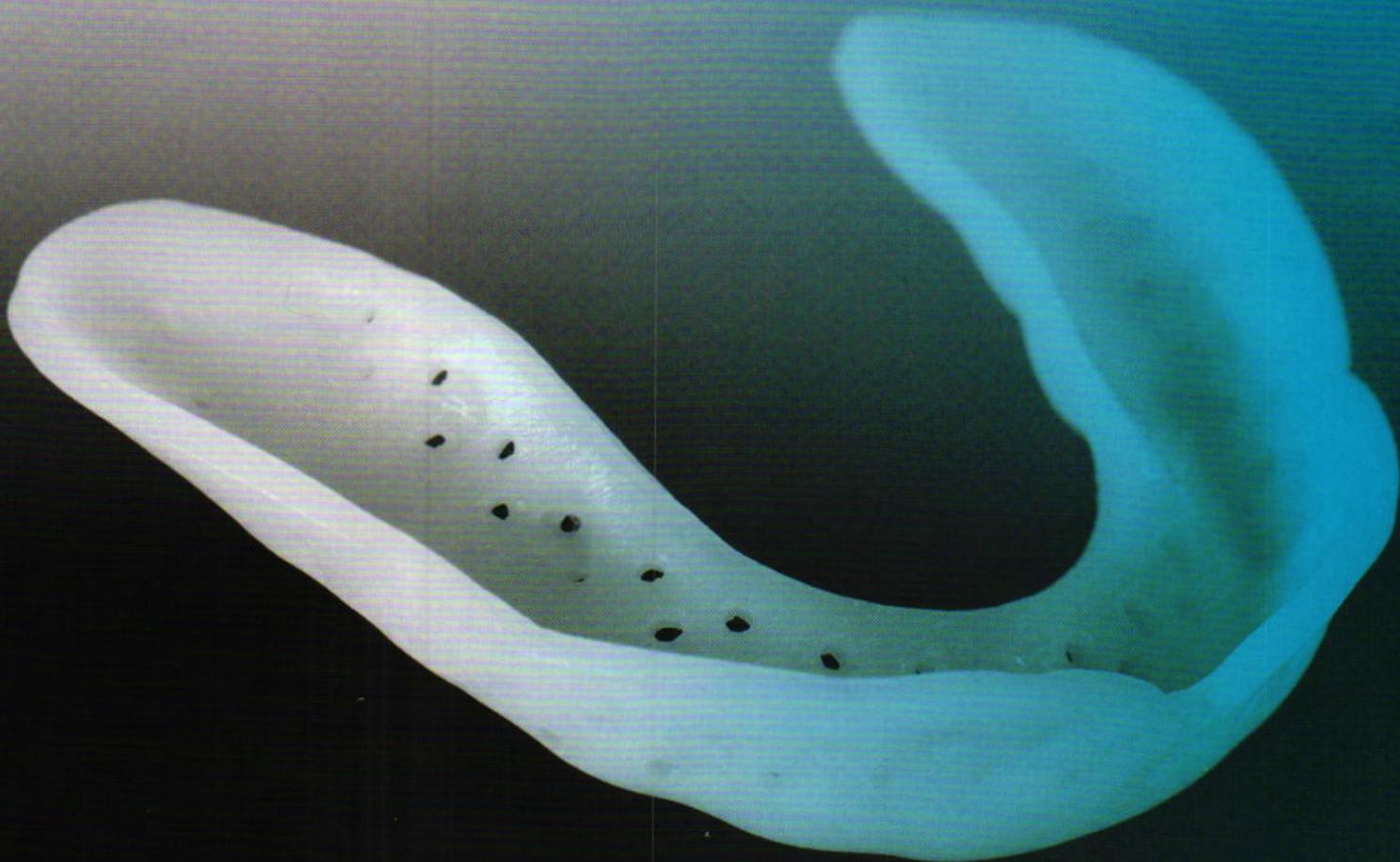

sovanightguard.com

 Diffusix[™] Technology  Remoldable  No Impressions  Custom Fit Comfort

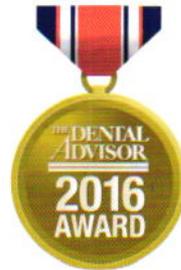

As Featured in *The Dental Advisor*.

Consultants' Comments

"Great option when we need an immediate splint."
"Molds and fits better than 'boil-and-bite' guards."
"The ability to be reheated and re-adapted numerous times is one of the best features."

Clinical Tip

While SOVA Night Guard is intended for use on the upper arch, it can be formed over the lower teeth if desired.

Description

SOVA Night Guard is a bruxism appliance designed to be formed by the patient at home. The finished guard is a tough polymer material with a flat occlusal surface and perforated labial and lingual flanges. The thermoplastic material has a high tensile strength and is biocompatible, biodegradable and BPA-free. The night guard starts as a flat, horseshoe-shape; after immersion in 140° water, the material becomes pliable. The night guard is then molded against the teeth until it becomes hardened, approximately one minute. SOVA Night Guard is only 1.6 mm thick, adding to its comfort, and can be heated and remolded up to 20 times. A 120-day durability warranty is provided. The SOVA Night Guard received a 81% clinical rating with The Dental Advisor.

Product Features


SOVA Night Guard provides a quick, easy option for protecting teeth from the effects of bruxism. The thin profile and perforated surface of SOVA Night Guard makes it comfortable for patients, even during the daytime.

The perforations serve three main functions:


1. Allow for the natural flow of air and saliva.
2. Aid in creating a true custom fit.
3. Oscillate on impact to diffuse grinding forces.

Consultants used SOVA Night Guard for the following circumstances: transitional dentition due to ongoing restorative treatment or growth, orthodontic patients, financial or time constraints, and as a trial appliance. Forming the splint is not difficult, but a successful outcome does require centering the soft material over the arch then pressing the lips and tongue against it. Often a second attempt was necessary after reheating the splint. Dentists would have preferred a thicker occlusal surface for durability and increased opening. When a lab-fabricated splint is not an option, SOVA Night Guard can provide occlusal protection from grinding.

Key Features

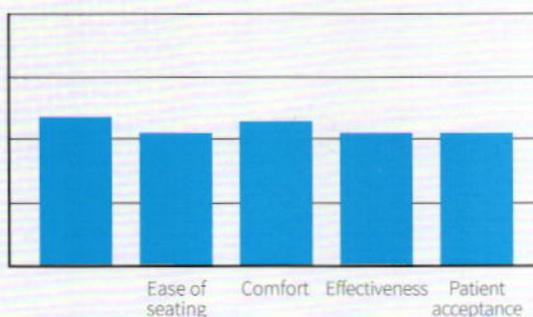

Percentage of Consultants who would:

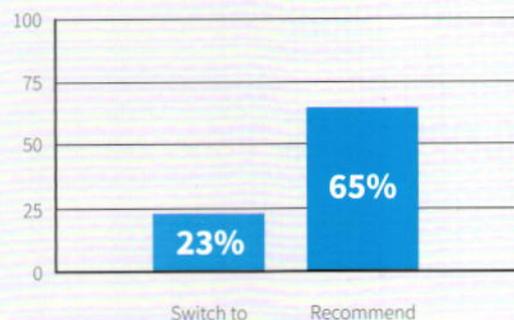

Dental Advisor - Top Products & Preferred Product Awards

"The Dental Advisor awards are one of the most respected awards created to recognize quality products and equipment, taking into account clinical and evidence-based research to honor the best."


Nighttime Grind Protection.TM

How it Works

Equipped with patented DiffusixTM Technology, SOVA Night Guard works with unique perforations and scientifically engineered "crumple zones" which prevent grinding forces from traveling to teeth. When SOVA is properly fitted, perforations oscillate and dissipate grinding forces into the crumple zones relieving pain and reducing the risk of dental injury.

So, why the perforations?

Designed for better durability and comfort, perforations allow for a custom fit and natural flow of air and saliva, enabling patients to drink without removing SOVA. Built on the basis of bridge engineering, perforations oscillate during night grinding and spread the impact forces throughout the surface of the guard and away from teeth.

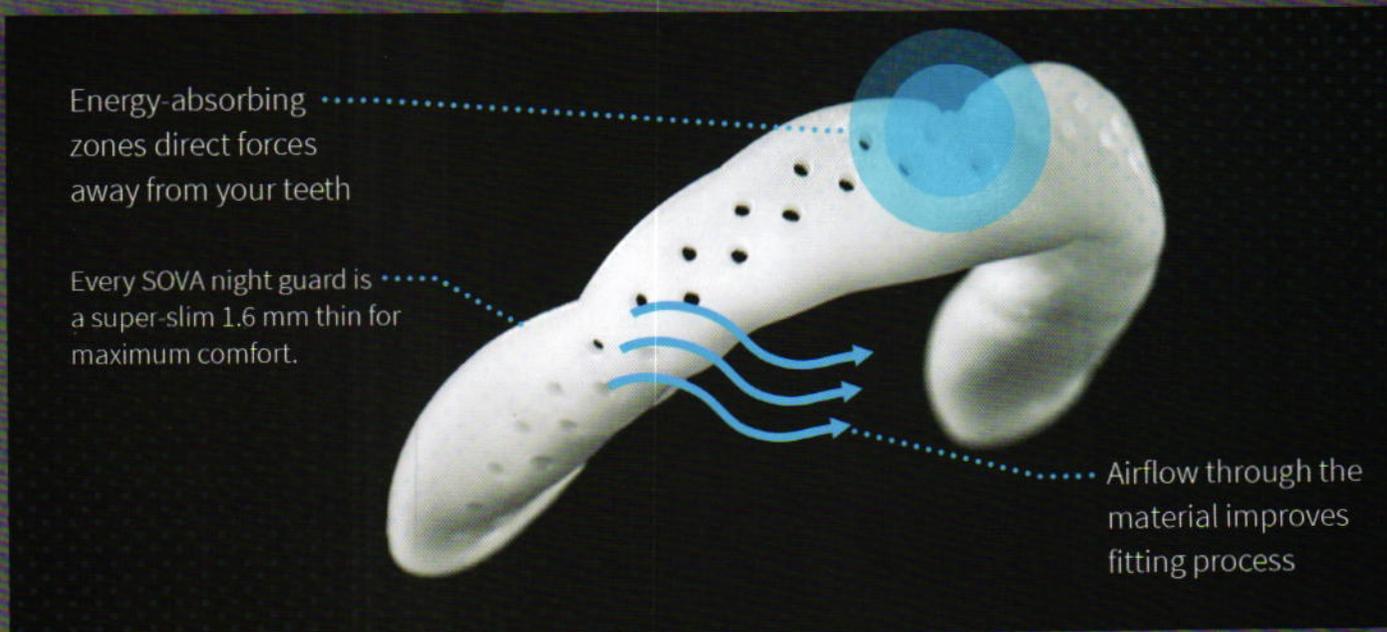

Advantages

- Custom comfort fit in minutes
- Natural flow of air and saliva
- Remoldable for up to 20 times
- Diffusix TechnologyTM protection
- Talk and Drink with ease
- Works with braces
- No Impressions
- 120 day durability warranty