

What steps are you taking to prepare your office for temporary closure?

Prepare your Cavitron® Ultrasonic Scaling System's waterline.

Follow these simple steps:

1. Turn off the dental office water supply.
2. Disconnect the Cavitron water supply line from the dental office water supply.
3. If a quick-disconnect connector is attached to the end of the hose, relieve the water pressure by pressing the tip of the connector into an appropriate container and allow water to drain.
4. Remove the hose from the Cavitron system, by pushing on the outer ring of the system's water inlet and gently pull out the water line.
5. As an additional precaution, blowing air through the waterline (using an air/water syringe) may also be helpful when preparing for an office closure and/or extended inactive use of a Cavitron Ultrasonic Scaling System.



Please refer to the "Water Line Connection" and "System Care" sections of your Cavitron Ultrasonic unit Instructions for Use (IFU) for additional information.

Want to learn more?

Watch this video on shutting down your Cavitron System:

<https://www.youtube.com/watch?v=5Af6vDVun2Q>

© 2020 Dentsply Sirona Preventive
ANN08-0320-1 (0320)