



The one visit, custom fit chairside Nightguard for dreamy results.



iNTERRA™

IN office Nightguard



- Lab-quality without an impression in only one visit
- Customized for perfect fit and patient comfort
- Highly durable and wear resistant

Call 1-800-LD-CAULK or go to
www.dentsply.com / www.caulk.com



iNterra™ Nightguard from DENTSPLY Caulk for the way YOU do dentistry.

iNterra™ Nightguard Process

*The entire process
is completed in your
office in one visit
in less than an hour.*



*Available in 3
individual tray sizes*

The one visit, custom fit, va



Form the iNterra™ Nightguard arch on lightly moistened teeth.

Tack cure in mouth with handheld light on the facial, lingual, and occlusal areas for one minute each with Quartz Halogen light (total 3 min) or 1.5 min each with LED Light (total 4.5 min).



Unique wedge shape allows for easier lateral excursions



The Opportunity

Over 45 million Americans suffer from Bruxism/Temporomandibular Disorder (TMD) but aren't treated with a nightguard.

Opportunity:


- Average dentist does 36-48 nightguards per year
- Only 3.6MM nightguards are laboratory-fabricated each year

Your practice can benefit from this under served population.

iNTERRA™

IN office Nightguard

Value-packed Nightguard for dreamy results.

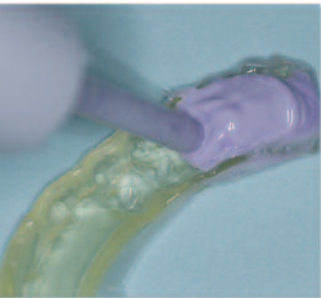

Remove appliance and reinforce with Regisil® Rigid Registration Material. Cure time is 2 min.

Cure extraorally (10 min cure cycle) in the iNterra™ VLC curing unit, or (20 min cure cycle) Triad® 2000 VLC Curing Unit (DENTSPLY Trubyte).

Test fit, adjust if necessary, polish, disinfect & deliver.

Doctors will like the iNterra™ Nightguard advantages:

- Easy-to-use
- Less expensive than conventional nightguards
- Improved fit over laboratory nightguards
- Requires fewer adjustments
- Improved patient compliance
- One visit delivery, preferably in combination with other services
- Highly durable and wear resistant

Patients will like that iNterra™ Nightguard:

- Looks good – highly esthetic
- Stays in place & is comfortable
- Fits teeth
- Is done in ONE visit...complete!
- Is Methyl Methacrylate Monomer free


The Science

iNterra™ Nightguard - A uniquely tough, protective yet comfortable VLC material.

iNterra™ VLC Material is uniquely rigid yet has a subtle elasticity to cushion the teeth.

Durability: Work of Fracture


All samples soaked 7 days at 37°C + 1 hour at 23°C prior to testing

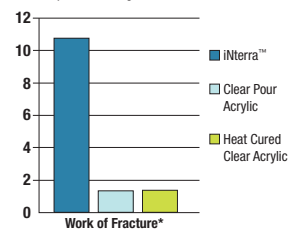

* Amount of energy spent to cause fracture or break
Conclusion: It would take a lot of chewing to break iNterra™

Wear Resistance

Leinfelder 3 Body Cycle Abrasive Wear Machine
200,000 Cycles at 50rpm

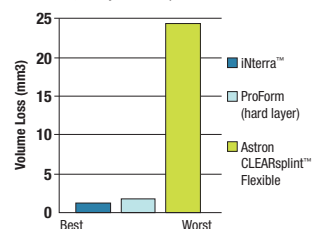


Dentists' evaluation results:

- ... the iNterra™ Material was easy to use (88%)
- ... the final device was esthetic (96%)
- ... the iNterra™ Nightguard fit well (92%)
- ... they would purchase iNterra™ Nightguard (72%)

iNterra™ VLC Curing Unit

Fabricate great fitting iNterra™ Nightguards fast and in-office

- Cure appliances in 10 minutes
- Processing capability for fixed & removable applications
- Built-in Smart Card expandability for future processing
- 2-year limited warranty

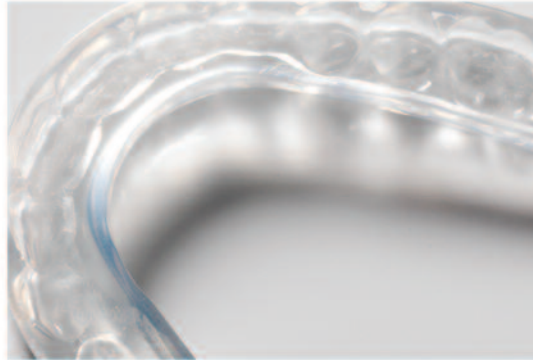

Cure Time: iNterra™ VLC Curing Unit: 10 min. (Triad VLC Curing Unit: 20 min.)

Ordering Information

iNterra™ INoffice Nightguard 61D010 Starter Kit

- 1 Small Arch
- 2 Medium Arches
- 1 Large Arch
- 1 Regisil® Rigid Super Fast Set cartridge (50mL)
- 6 Mixing Tips



Refill Package

- 6 iNterra™ Nightguard Arches
- 61D020 Small Arch Refill Package
- 61D030 Medium Arch Refill Package
- 61D040 Large Arch Refill Package



61D100 iNterra™ VLC Curing Unit – 120V

91D400 iNterra™ Replacement Bulbs – 3-pack

51D901 Patient Placard – Bruxism

51D902 Patient Poster – Bruxism



51D904 Patient Placard –
Cosmetic Dentistry

51D905 Patient Poster –
Cosmetic Dentistry




Regisil® Rigid Bite Registration Material

619420 Intro Kit
4 Cartridges (50mL each)
12 Micro-Mixing Tips
1 Cartridge Dispenser

619425 4-pack Cartridge Refill
4 Cartridges (50mL each)
12 Micro-Mixing Tips

619451 50mL Micro-Mixing Tips Refill Package
12 Micro-Mixing Tips for Regisil® Rigid
50mL cartridge

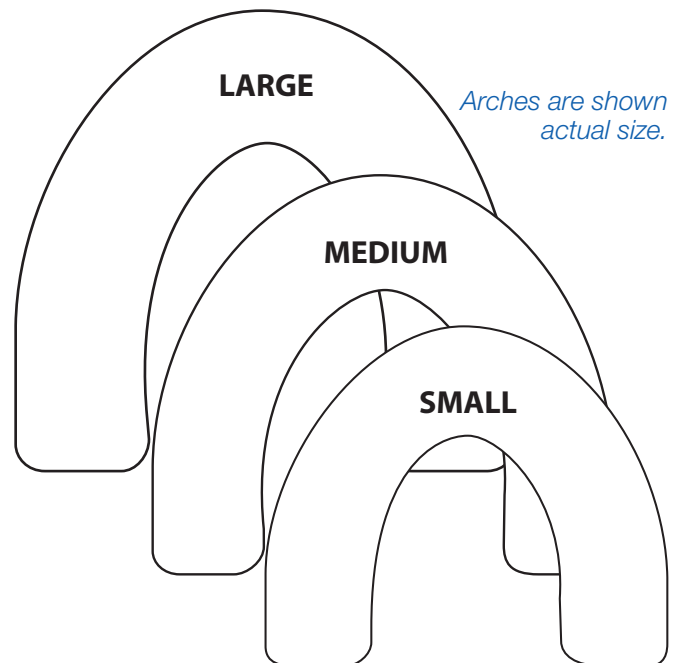

DENTSPLY
CAULK

38 West Clarke Avenue
Milford, DE 19963 USA
www.dentsply.com

www.caulk.com
1-800-LD-CAULK
In Canada, 1-800-263-1437

All data on file.
ProForm and Astron CLEARsplint Flexible are not registered trademarks of DENTSPLY International.
©2011 DENTSPLY International. All rights reserved. Form #51D900 (R 1/26/11)

